

## Directions for use:

Remove the thin film from the adhesive side of the sheeting.

Remove the thick plastic film from the non-sticky side and retain it for use as a covering for the sticky side when the sheeting is not in use.



Apply the re-usable sheeting, sticky side to the skin, for 8 to 12 hours daily and continue treatment until desired results or up to 12 weeks

The sheeting can be cut to size and shape as necessary

Once a week clean the sheeting in warm, soapy water

There are over 50 studies proving the efficacy of Silicone Gel Sheet-ing. Scar Fx® is a medically proven, non-surgical, low cost approach to soften, smooth and reduce your scar. Scar Fx® is effective on even on the most severe scars. It neutralises the scar's colour and works to flat-ten raised scars. It is easy to use and only one sheet is required to treat most scars. Manufactured in the United States, Scar Fx® is a medical grade, adherent silicone that is soft, pliable and contours easily to any part of the body. Medically proven effective to reduce your scar without surgery.



After 1 week  
Scar raised  
and pronounced



After 8 weeks  
Scar softened  
flattened and fading



After final  
treatment